

Initiating the Neo-Laser.

Step-1.

Insert the interlock into the rear of the machine. Insert into the port located beneath the **RED** indicator



Step-2.

Release/Open the Emergency Stop Switch by pulling outwards from the console. It will move outwards only 3-4mm.

Emergency Stop Engaged Position



Emergency Stop Open Position
(required to operate the laser)



Remove Dust Cover on the Front Panel




Plug in the Power Chord (on the right side), and the Activation Pedal (left side, blue hub).

The Activation Pedal has a red dot which must face up (twelve o'clock position).

You do not need to twist or turn either cable to connect or disconnect.

Turn on the Power.

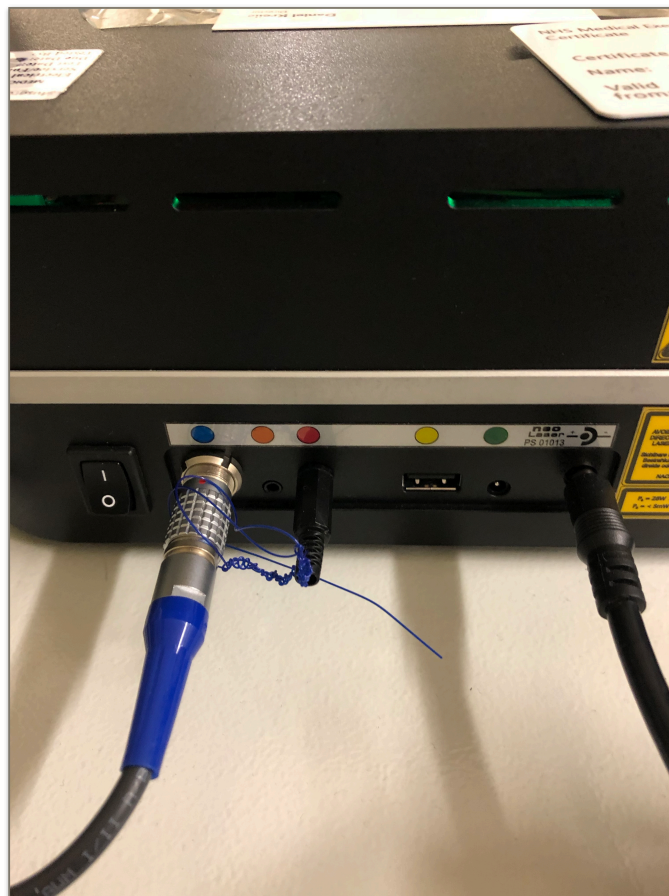

Allow up to a minute for the laser to initiate and self calibrate. A progress-bar will show along the bottom of the screen during the initiation process.



Once initiated, enter the password. (The factory default is 0000)



Progress through the programmed treatments to arrive at the main operating window. The power, on-time, and off time can all be changed by touching the parameter you want to change. Then the corresponding arrow keys.



Plug the laser fibre into the fibre port. The 'notch in the hub must be facing down (six o'clock orientation)



A green indicator light will show on the top left corner of the laser, indicating safe. To operate the laser, press the Standby key on the front screen.


The Standby button will change from Green to Yellow, and now read Ready.

The indicator light will turn to the colour yellow. The laser is now armed. Once the operator stands on the pedal, the indicator light will turn red, and commence firing the laser.



Once finished with the laser, the total joules should be recorded along with the centimetres treated.

To reset the joules counter, simply press the value.

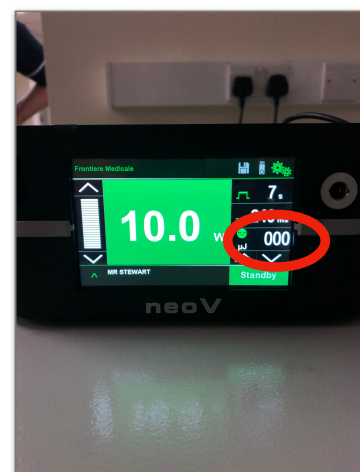

— end —